



**PHSMedical**  
by Pivotal Health Solutions

# Electric Epidural Positioning Device

## Quick Start Guide



## QUESTIONS?

[phsmedicalsolutions.com](http://phsmedicalsolutions.com)

800-743-7738



# Electric Epidural Positioning Device Quick Start Guide

**1) Position patient in an upright sitting position at the edge of the bed**



**2) Roll the EPD into place and lock the casters**



**3) Using the UP/DOWN buttons on the hand control, adjust the EPD to proper height**



**4) Have the patient lean forward, resting on face rest and arm rest**



**5) Adjust the Face Rest to desired position**



**6) Adjust the Arm Rest by pressing toggle switch and sliding into position - release to lock**



**7) Using BOTTOM UP/DOWN buttons on hand control, adjust the foot rest**



**8) Adjust chest support by pressing down toggle switch and sliding with gentle pressure - making a nice curve of the back**



**9) After the procedure, have patient lean back into a normal sitting position**



**10) Unlock the casters by lifting up on brake lever, roll the EPD away from patient**

